## ZAMST A2-DX (ANKLE SUPPORT) INSTRUCTIONS FOR USE

EN

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

### Read this first:

- This product is intended for stabilizing the ankle.
- This product is side (left and right) specific.
- Choose correct size and apply the product properly as instruction, otherwise you may not obtain proper performance of the product.

### When using the product:

- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.
- When storing the product, avoid applying excessive weight to the hook-and-loop fasteners or raised fabric. Otherwise, the hook-and-loop fasteners may not work effectively.
- Do not wear the product while driving. Doing so may have an adverse effect on your driving ability.

### Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

**⚠ WARNING:** Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

**⚠ CAUTION:** Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

### ⚠ WARNING

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
  - Those predisposed to allergies or with sensitive skin.
  - Those who have a fracture, cut, numbness, swelling, eczema, rash or other abnormalities in the area where the product will be worn.
- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
  - The product should only be used on ankles.
  - Be sure to use the correct size product.
  - Do not use the product while sleeping.
  - If wearing the product for a long period of time, readjust it as necessary. The application of constant pressure for a long period of time may cause skin and/or circulatory problems.
  - When using the product, be sure to put it on in the correct orientation (top/bottom, inside/outside, front/back).
  - Follow the attachment steps in order to use this product correctly.
  - Do not wrap the main body of the product or the straps too tightly.
  - If the product loosens or comes off during use, be sure to refasten it correctly by following the noted steps from the beginning.
  - If the product shows any signs of wear or damage, or a hook-and-loop fastener no longer works well, do not use the product.
  - Do not attempt to modify or repair the product.
- The original guard of the product is made of plastic. To prevent personal injury, do not use the product in the following situation:
  - Where contact with the product may injure others.
- If any of the following abnormalities occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms:
  - If this product causes discomfort.
  - If an injury, fracture, cut, pain, numbness, swelling, eczema, rashes, etc. occurs in the area where the product is worn.

### ⚠ CAUTION

- Hard materials are used for the hook-and-loop fasteners. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
- Be sure to wear the product over a sock. Do not wear it directly on your skin.
- Sweat and friction may cause the color of this product to fade or be transferred to other materials.
- Make sure the hook-and-loop fasteners not attach to clothing. If they do, clothing may become frayed or be damaged.

## Care and cleaning

### ⚠ CAUTION



- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.
- If washing the product together with clothing, etc., be sure to use a laundry net. If a hook-and-loop fastener becomes attached to clothing, it may cause the clothing to become frayed or be damaged.

- The product is hand washable. Gently press wash and pat dry using a towel, and line dry in the shade.
- When washing this product, attach the hook-and-loop fasteners to the main body. Attempting to wash this product without closing the hook-and-loop fasteners may weaken its fastening capability.
- Do not use an iron or chlorine bleach, however, oxygen bleach can be used.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.

# How to apply

- Before attaching this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly attached.
- The steps below use the right foot for reference. Follow the same steps for the left foot.



1. Unfasten all hook-and-loop fasteners of the product. Confirm the top/bottom, inside/outside, and front/back of the product.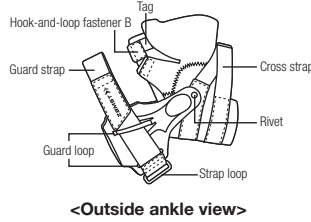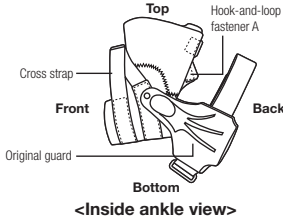

Make sure that the guard strap is passed through the guard loop.



\* You do not need to detach the guard strap from the guard loop when attaching or removing the product.

2. Position your ankle at right angle, open the rear entry of the product and slide your foot into the opening from the back.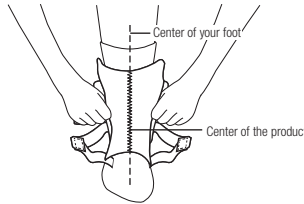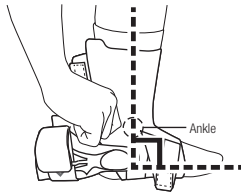



With the original guards folded to the back, slide your foot into the product from the back until your ankle is aligned with the area indicated by the dashed line which is directly above the rivet on the original guard.

3. Fasten hook-and-loop fastener A.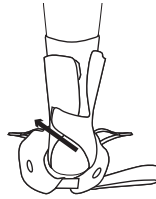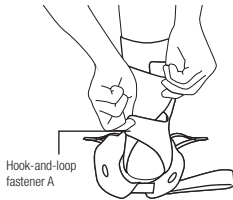


Hold hook-and-loop fastener A with your left hand and hold the product end with your right hand so that they crisscross. Place your feet on the floor with your ankle at a 90 degree angle. Wrap the support around your ankle by pulling hook-and-loop fastener A diagonally and fastening it.

4. Fasten hook-and-loop fastener B.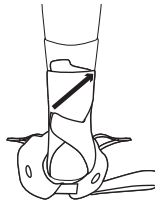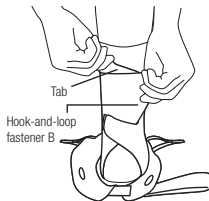


Hold hook-and-loop fastener B with your right hand and hold the tab with your left hand so that they crisscross. Wrap the product around your ankle by pulling hook-and-loop fastener B diagonally and fastening it.

5. Fasten the cross straps.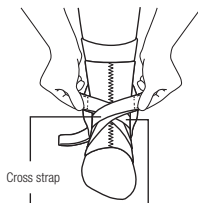

Cross both cross straps while pulling them in opposing directions in front of your foot and attach the hook-and-loop fasteners above the ankle on opposite sides.


6. Attach the original guards to the main body of the product.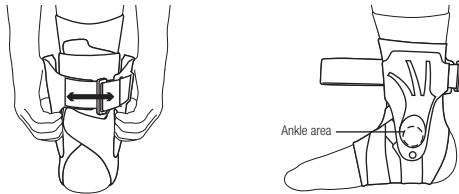

**Correct position of guard**

Raise the original guards in the upright position. Position them so that the raised area on the guard (directly above the rivet) comes over the ankle indicated by the dashed circle by adjusting the length of the rear position of the upper strap. Fasten the hook-and-loop fasteners on the inner side of each original guard to the main body of the product.

7. Wrap the guard straps.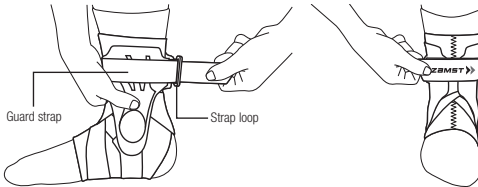


Use one hand to prevent the guards from moving. Wrap the guard strap around your ankle and pass the strap through the strap loop. Fold the strap backward and fasten it with the hook-and-loop fastener.

\* Do not overtighten the straps as doing so may cut off blood circulation to the foot or lead to skin and/or circulatory problems.

8. Make sure that the product is not loose or out of place. If you feel any discomfort, repeat the steps from the beginning to reattach the product.


**The product correctly fastened (right foot)**

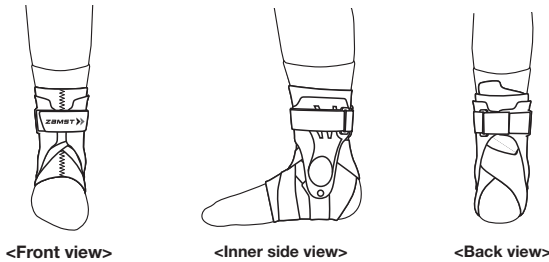\* In the illustration, the logo is facing forward. The logo does not always come in the center. Its position may differ with each individual user.

\* Adjust the strap bearing the logo as necessary to ensure a snug fit.

Two soft spacers are provided with the product. Attach them if the original guards or rivets are pressed against your feet and are making you uncomfortable.

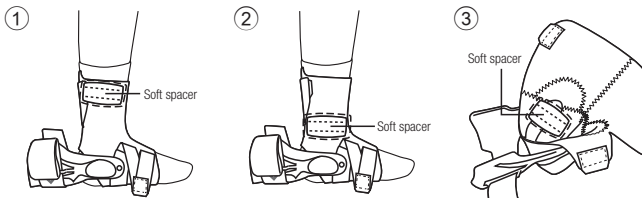- ① If the location being pressed is on the upper part of your foot, attach the soft spacer on the upper part of the guard.
- ② If the location being pressed is on your ankle, attach the spacer on the upper part of your ankle.
- ③ If a rivet is pressing against your foot, attach the spacer inside the plate.

**Reporting Obligation**

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

**Disclaimer:**

- \* The product specifications and appearance are subject to change without notice for the purpose of improvement.
- \* When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.
- \* We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmax Co., Ltd.

"ZAMST" and "ZAMST logo" are trademarks or registered trademarks of Nippon Sigmax Co., Ltd in Japan, the United States and other countries.

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