The video on how to apply the product, and the Instructions for Use in other languages are available at

<https://zamst.com/products/iw-1-set/>

<https://zamst.com/products/iw-2-set/>

<https://zamst.com/products/icebag/>

## ZAMST ICE BAG AND ICING WRAP INSTRUCTIONS FOR USE

EN

For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

### Read this first:

- These instructions for use are applicable to Ice Bag, IW-1, and IW-2.
- This product is intended for cooling and compressing areas such as arms and legs.
- Choose correct size and apply the product properly as instruction, otherwise you may not obtain proper performance of the product.

### When using the product:

<Ice Bag, IW-1, and IW-2>

- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.

<Ice Bag>

- Putting ice with sharp edges into the ice bag may damage the inner coating of the ice bag, causing water to leak out. Crush ice or add some water to smoothen any sharp edges before putting it into the bag.

<IW-1 and IW-2>

- When storing the product, avoid applying excessive weight to the hook-and-loop fasteners or raised fabric. Otherwise, the hook-and-loop fasteners may not work effectively.

### Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

- ⚠ WARNING:** Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

- ⚠ CAUTION:** Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

### ⚠ WARNING

<Ice Bag, IW-1, and IW-2>

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
  - Those predisposed to allergies or with sensitive skin.
  - This product contains natural rubber. Natural rubber contains latex allergens and may cause allergic reactions such as itching, hives, or difficulty breathing. If any of the above-mentioned symptoms are observed, immediately stop using this product and seek medical attention.
  - Those who have cuts, numbness, eczema, rashes or other abnormalities in the area where the product will be worn.
- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
  - The product should only be used for holding the cooling materials, and cooling and compressing the desired area.
  - Do not use the product while sleeping.
  - If the product shows any signs of wear or damage, or a hook-and-loop fastener no longer works well, do not use the product.
  - Do not attempt to modify or repair the product.
- If any abnormalities, such as an injury, pain, numbness, eczema, etc. occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms.

<IW-1 and IW-2>

- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
  - Be sure to use the correct size product.
  - Follow the attachment steps in order to use this product correctly.
  - Do not wrap the main body of the product or the straps too tightly.
  - If the product loosens or comes off during use, be sure to refasten it. If wearing the product for a long period of time, take extra care to prevent circulatory problems and frostbite, and readjust the tightness as necessary.

### ⚠ CAUTION

<Ice Bag, IW-1, and IW-2>

- Sweat and friction may cause the color of this product to fade or be transferred to other materials.

<Ice Bag>

- Be sure to close the cap tightly after putting ice or water into the bag, otherwise the cap may come loose during use, which may result in water or ice leaking out, which may cause your clothing or body to become wet.
  - Condensation may cause clothing or the area being cooled to become wet.
- <IW-1 and IW-2>
- Hard materials are used for the hook-and-loop fasteners. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
  - Make sure the hook-and-loop fasteners not attach to clothing. If they do, clothing may become frayed or be damaged.

## Care and cleaning

### ⚠ CAUTION


- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.
- If washing the product together with clothing, etc., be sure to use a laundry net in order to prevent the clothing from becoming frayed or be damaged.

<Ice Bag>

- The product is hand washable. Press wash and pat dry using a towel, and line dry in the shade.
  - Do not use an iron, chlorine bleach, or oxygen bleach. Doing so may damage the product.
  - Do not tumble dry. Doing so may damage the product.
  - This product cannot be dry or wet cleaned.
- <IW-1 and IW-2>
- This product can be washed in a washing machine. After washing, hang it to dry in the shade.
  - When washing this product, attach the hook-and-loop fasteners to the main body. Attempting to wash this product without closing the hook-and-loop fasteners may weaken its fastening capability.
  - Do not use an iron or chlorine bleach, however, oxygen bleach can be used.
  - Do not tumble dry. Doing so may damage the product.
  - This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.

## How to use Ice Bag

- Before using this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly used.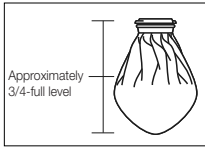


1. Fill the ice bag 3/4 full with ice. Adding some water to the bag will help it conform to the shape of your body.
2. Let as much air out as possible before screwing the cap on tightly.
3. Apply the ice bag to the desired area.

## How to apply


- Before attaching this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly attached.

### How to apply IW-1

1. Put the ice bag(s) inside the ice bag pocket and pull it into shape.
2. Attach the product to the area you want to cool, and fasten the temporary hook-and-loop fastener to the product. If the temporary fastening strap feels excessively tight, you do not need to attach it. If you are not using the temporary fastening strap, fasten the temporary fastening hook-and-loop fastener to the napping fabric for the temporary fastening strap.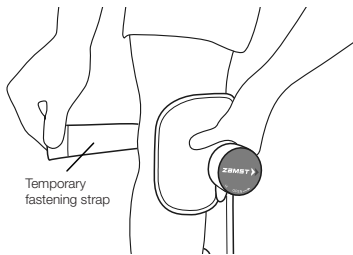

3. Wrap the straps. Make sure that the icing wrap is not loose or misaligned. If you feel any discomfort, repeat the steps from the beginning to reattach the product.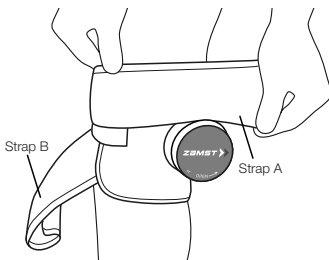

### Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

### Disclaimer:

- \* The product specifications and appearance are subject to change without notice for the purpose of improvement.
- \* When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.
- \* We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmex Co., Ltd.


"ZAMST" and "ZAMST logo" are trademarks or registered trademarks of Nippon Sigmex Co., Ltd in Japan, the United States and other countries.

### How to apply IW-2

- Put the ice bag(s) inside the ice bag pocket(s) and pull it into shape. There are three ice bag pockets. Put the ice bag(s) in the pocket(s) so that the ice bag(s) is appropriately positioned for the area you want to cool.

#### ● Attaching the product to your shoulder


1. Attach one end of the hook-and-loop fastener of the fix strap provided together with the product to the approximate location indicated in the illustration shown below.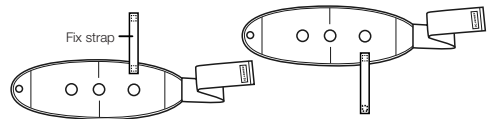

<When attaching to right shoulder>

<When attaching to left shoulder>

2. Attach the product to the part of your shoulder you want to cool while holding the product with the finger hole on the front. Hook your finger into the finger hole. Pull the elastic strap with that finger and wrap it around your body, then fasten it.

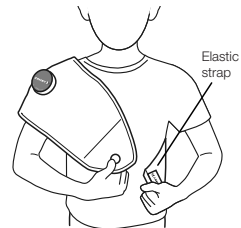
3. Pull the fix strap and attach it to the front of the icing wrap. Make sure that the icing wrap is not loose or misaligned. If you feel any discomfort, repeat the steps from the beginning to reattach the product.

#### ● Attaching the product to your lower back

Attach the icing wrap to the area you want to cool. Pull the elastic strap and fasten the hook-and-loop fastener. The fix strap provided with the product is not used.

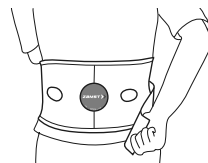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