## ZAMST JK-1 (KNEE SUPPORT) INSTRUCTIONS FOR USE (For left or right side)

EN

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

### Read this first:

- This product is intended for reducing strain on the bottom part of the Patella.
- This product can be used for both left and right sides.
- Choose correct size and apply the product properly as instruction, otherwise you may not obtain proper performance of the product.

### When using the product:

- Exercise care to ensure that the hook-and-loop fasteners do not come into contact with the mesh fabric on the back of the knee. The hook-and-loop fasteners may damage the mesh fabrics.
- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.
- When storing the product, avoid applying excessive weight to the hook-and-loop fasteners or raised fabric. Otherwise, the hook-and-loop fasteners may not work effectively.

### Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

**⚠ WARNING:** Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

**⚠ CAUTION:** Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

### ⚠ WARNING

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
  - Those predisposed to allergies or with sensitive skin.
  - Those who have a fracture, cut, numbness, swelling, eczema, rash or other abnormalities in the area where the product will be worn.
- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
  - The product should only be used on knees.
  - Be sure to use the correct size product.
  - Do not use the product while sleeping.
  - If wearing the product for a long period of time, readjust it as necessary. The application of constant pressure for a long period of time may cause skin and/or circulatory problems.
  - When using the product, be sure to put it on in the correct orientation (top/bottom, inside/outside, front/back).
  - Follow the attachment steps in order to use this product correctly.
  - Do not wrap the main body of the product or the straps too tightly.
  - If the product loosens or comes off during use, be sure to refasten it correctly by following the noted steps from the beginning.
  - If the product shows any signs of wear or damage, or a hook-and-loop fastener no longer works well, do not use the product.
  - Do not attempt to modify or repair the product.
- If any of the following abnormalities occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms:
  - If this product causes discomfort.
  - If an injury, fracture, cut, pain, numbness, swelling, eczema, rashes, etc. occurs in the area where the product is worn.

### ⚠ CAUTION

- Hard materials are used for the hook-and-loop fasteners. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
- Sweat and friction may cause the color of this product to fade or be transferred to other materials.
- Make sure the hook-and-loop fasteners do not attach to clothing. If they do, clothing may become frayed or be damaged.

## Care and cleaning

### ⚠ CAUTION


- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.
- If washing the product together with clothing, etc., be sure to use a laundry net. If a hook-and-loop fastener becomes attached to clothing, it may cause the clothing to become frayed or be damaged.

- This product can be washed in a washing machine. After washing, hang it to dry in the shade.
- When washing this product, attach the hook-and-loop fasteners to the main body. Attempting to wash this product without closing the hook-and-loop fasteners may weaken its fastening capability.
- Do not use an iron or chlorine bleach, however, oxygen bleach can be used.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.


## How to apply

- Before attaching this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly attached.
- The steps below use the right leg for reference. Follow the same steps for the left leg.

1. Unfasten all the hook-and-loop fasteners of the product. Confirm the top/bottom and inside/outside of the product.

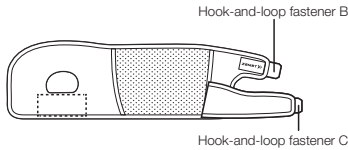
<Outside of the product>
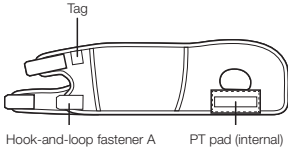
<Inside of the product>


Place the side with the size tag on the back of the product face up when attaching the product to your knee. Confirm the vertical positioning of the product by checking the direction of the arrow on the tag.

2. Attach the product to the front of your knee.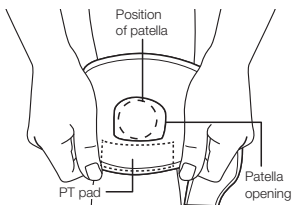


With the PT pad on the inside of the product, place the product on your knee so that your patella fits in the center of the patella opening. Be sure to place it on your knee so that the PT pad is touching the lower part of your patella (patellar ligament).

3. Fasten hook-and-loop fastener A while pulling the product.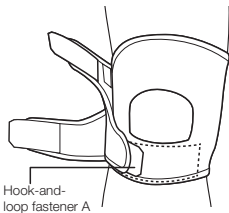


When doing so, make sure that the position of the PT pad does not shift.

4. Straighten your knee and fasten hook-and-loop fastener B.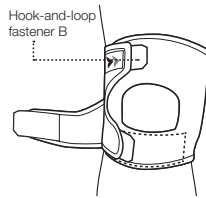


Fasten hook-and-loop fastener B for appropriate compression.

5. Fasten hook-and-loop fastener C while pulling the understrap.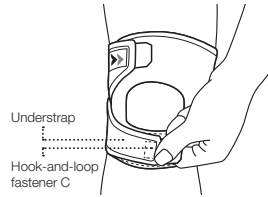

Fasten hook-and-loop fastener C while pulling the understrap so that the fastener goes on top of the PT pad below your knee.

6. Straighten your knee and pull the product up and down to eliminate slack.

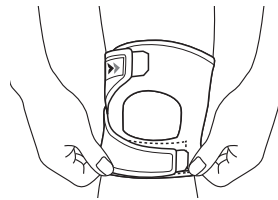
Pull up the slack on the back of your knee.

7. Make sure that the product is not loose or out of place. If you feel any discomfort, repeat the steps from the beginning to reattach the product.


#### The product correctly fastened (right leg)

<Front view>

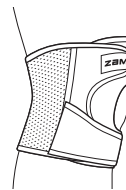<Side view>
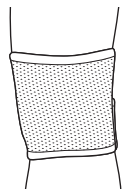
<Back view>

#### Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

#### Disclaimer:

- \* The product specifications and appearance are subject to change without notice for the purpose of improvement.
- \* When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.
- \* We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmax Co., Ltd.

"ZAMST" and "ZAMST logo" are trademarks or registered trademarks of Nippon Sigmax Co., Ltd in Japan, the United States and other countries.

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