## ZAMST SHOULDER WRAP (SHOULDER SUPPORT) INSTRUCTIONS FOR USE (For left or right side)

For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

### Read this first:

- This product is intended for stabilizing the shoulder.
- This product can be used for both left and right sides.
- Choose correct size and apply the product properly as instruction, otherwise you may not obtain proper performance of the product.

### When using the product:

- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.
- When storing the product, avoid applying excessive weight to the hook-and-loop fasteners or raised fabric. Otherwise, the hook-and-loop fasteners may not work effectively.

### Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

**⚠ WARNING:** Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

**⚠ CAUTION:** Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

### ⚠ WARNING

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
  - Those predisposed to allergies or with sensitive skin.
  - Those who have a fracture, cut, numbness, swelling, eczema, rash or other abnormalities in the area where the product will be worn.
- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
  - The product should only be used on shoulders.
  - Be sure to use the correct size product.
  - If you are pregnant or you think you are pregnant, consult a physician.
  - Do not use the product while sleeping.
  - Follow the attachment steps in order to use this product correctly.
  - Do not wrap the main body of the product or the straps too tightly.
  - If the product loosens or comes off during use, be sure to refasten it correctly by following the noted steps from the beginning. If wearing the product for a long period of time, take extra care to maintain appropriate compression force, and readjust the tightness as necessary.
  - If the product shows any signs of wear or damage, or a hook-and-loop fastener no longer works well, do not use the product.
  - Do not attempt to modify or repair the product.
- If any abnormalities, such as an injury, fracture, pain, numbness, swelling, eczema, etc. occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms.

### ⚠ CAUTION

- Hard materials are used for the hook-and-loop fasteners. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
- Be sure to wear the product over an undershirt or similar garment instead of directly on your skin.
- Sweat and friction may cause the color of this product to fade or be transferred to other materials.
- Make sure the hook-and-loop fasteners not attach to clothing. If they do, clothing may become frayed or be damaged.

## Care and cleaning

### ⚠ CAUTION



- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.
- If washing the product together with clothing, etc., be sure to use a laundry net in order to prevent the clothing from becoming frayed or be damaged.

- This product can be washed in a washing machine. After washing, hang it to dry in the shade.
- When washing this product, attach the hook-and-loop fasteners to the main body. Attempting to wash this product without closing the hook-and-loop fasteners may weaken its fastening capability.
- Do not use an iron or chlorine bleach, however, oxygen bleach can be used.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.


# How to apply

- Before attaching this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly attached.
- The steps below use the right shoulder for reference. Follow the same steps for the left shoulder.

1. Unfasten all the hook-and-loop fasteners of the product. Confirm the top/bottom and inside/outside of the product. Place the side with the size tag on the back of the product face up when attaching the product to your shoulder. Place the back side of the chest strap with the blue fabric on your body.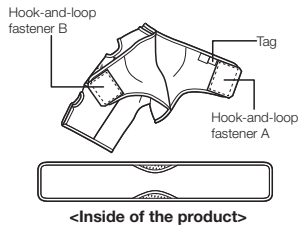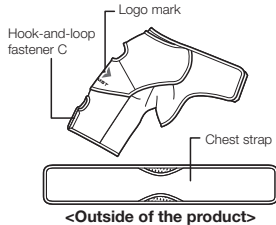


2. Attach the chest strap to hook-and-loop fastener A (with a size tag) on the product.

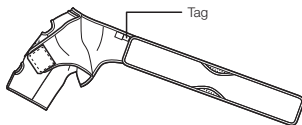
\* When attaching the product to your left shoulder, attach the chest strap to hook-and-loop fastener B.


3. Pull the product up until the acromion (a large bony projection) of your shoulder is positioned under the hole of the product.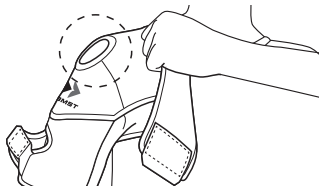

4. Pull the chest strap from the back to the front through under your armpit, and fasten hook-and-loop fastener B over your chest to apply appropriate compression. Adjust the position of the chest strap until its center is under your armpit. When attaching the product to your left shoulder, fasten hook-and-loop fastener A over your chest.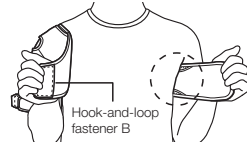

5. Fasten hook-and-loop fastener C so that the product fits the upper arm.

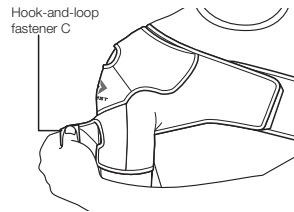6. Pull the product up and down to eliminate slack.

7. Make sure that the product is not loose or out of place. If you feel any discomfort, repeat the steps from the beginning to reattach the product.

## Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

## Disclaimer:

- \* The product specifications and appearance are subject to change without notice for the purpose of improvement.
- \* When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.
- \* We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmex Co., Ltd.

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