ZAMST THUMB GUARD (THUMB SUPPORT) INSTRUCTIONS FOR USE (For left or right side)

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

Read this first:

- This product is intended for stabilizing the thumb.
- This product can be used for both left and right sides.
- Choose correct size and apply the product properly as instruction, otherwise you may not obtain proper performance of the product.

When using the product:

- Do not leave the product in locations subject to high temperatures, such as on a car dashboard or locations subject to direct sunlight. Doing so may cause the product to deform or degrade.
- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.
- When storing the product, avoid applying excessive weight to the hook-and-loop fasteners or raised fabric. Otherwise, the hook-and-loop fasteners may not work effectively.

Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

⚠ WARNING: Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

⚠ CAUTION: Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

⚠ WARNING

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
 - Those predisposed to allergies or with sensitive skin.
 - Those who have a fracture, cut, numbness, swelling, eczema, rash or other abnormalities in the area where the product will be worn.
- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
 - The product should only be used on thumbs.
 - Be sure to use the correct size product.
 - Do not use the product while sleeping.
 - If wearing the product for a long period of time, readjust it as necessary. The application of constant pressure for a long period of time may cause skin and/or circulatory problems.
 - When using the product, be sure to put it on in the correct orientation (top/bottom, inside/outside).
 - Follow the attachment steps in order to use this product correctly.
 - Do not wrap the main body of the product or the straps too tightly.
 - If the product loosens or comes off during use, be sure to refasten it correctly by following the noted steps from the beginning.
 - If the product shows any signs of wear or damage, or a hook-and-loop fastener no longer works well, do not use the product.
- Be sure to correctly mold the product by adhering to the following requirements. Otherwise, serious injury, such as burns, may result:
 - Be sure to mold the product in a way that is described in the instructions for use.
 - If the product is to be used by a child, make sure that a parent or adult mold it.
 - When you use hot water, be careful not to burn yourself.
 - After putting the product in hot water, dry it well using a towel and make sure that it is not hot before attaching it to your hand.
- The thermal fit guard of the product is made of plastic. To prevent personal injury, do not use the product in the following situation:
 - Where contact with the product may injure others.
- If any of the following abnormalities occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms:
 - If this product causes discomfort.
 - If an injury, fracture, cut, pain, numbness, swelling, eczema, rashes, etc. occurs in the area where the product is worn.

⚠ CAUTION

- Hard materials are used for the hook-and-loop fasteners. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
- Sweat and friction may cause the color of this product to fade or be transferred to other materials.
- Make sure the hook-and-loop fasteners not attach to clothing. If they do, clothing may become frayed or be damaged.

Care and cleaning

⚠ CAUTION

- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.
- If washing the product together with clothing, etc., be sure to use a laundry net. If a hook-and-loop fastener becomes attached to clothing, it may cause the clothing to become frayed or be damaged.

- This product can be washed in a washing machine. After washing, hang it to dry in the shade.
- When washing this product, attach the hook-and-loop fasteners to the main body. Attempting to wash this product without closing the hook-and-loop fasteners may weaken its fastening capability.
- Do not use an iron or chlorine bleach, however, oxygen bleach can be used.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.

How to mold and apply


- The thumb guard is a product that needs to be molded to fit your hand and fingers using hot water before use.
- **Before attaching this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly attached.**
- The molding and attachment steps below use the thumb of your right hand for reference. Follow the same steps for the left hand.
- The guard contains a resin that softens at a temperature of approximately 70 °C (158 °F), and hardens when cooled.
- If the product loses its original shape during use, its shape can be corrected by re-molding it. The product can be re-molded approximately 20 times.

1. Prepare a container (such as a basin) that is large enough to put the product in and a towel for drying.

Be sure to use a heat-resistant container.

2. Fill the container with approximately 70 °C (158 °F) hot water.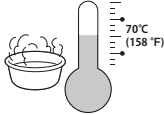

3. Unfasten all the hook-and-loop fasteners of the product. Confirm the top/bottom and inside/outside of the product.

<Outside of the product>

Place the side with the size tag on the back of the product facing down when attaching the product to your hand.

<Inside of the product>

4. Hold the support strap and the thumb strap with each hand and immerse the product in hot water for 15 to 20 seconds.


Be careful not to burn yourself by accidentally dipping your hand in the hot water.

5. Remove the product from the hot water and dry it off using a towel.

Wrap the product in a towel to dry it thoroughly.


6. Confirm that the product has cooled down so that you can touch it. Fold the product in half from the center.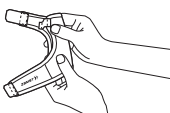

Be careful not to burn yourself.


* It takes about 60 to 90 seconds for the product to reach an optimal temperature after removing it from hot water.

7. Slip the finger loop over your thumb and position the product where you want to place it.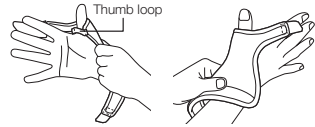


When attaching the product that has already been molded, please start the steps from here.

8. Fasten the support strap.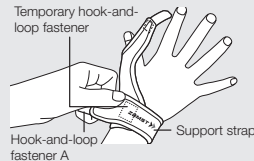

Fasten the support strap to the temporary hook-and-loop fastener while pulling the strap lightly, and fasten hook-and-loop fastener A to the product.


9. Fasten the thumb strap.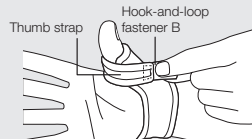

Fasten hook-and-loop fastener B while lightly pulling the thumb strap. To make it easier to put on a glove, etc. over the product, pull the strap downward and fasten.

* Perform the following only when molding the product.


Press the entire guard with your other hand to fit it to the shape of your hand and fingers, and wait until the guard hardens.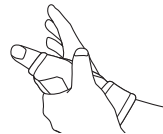

- * After removing the guard from the hot water, it hardens in 3 to 4 minutes.
- * If you want the guard to harden quickly, put it in cold water.
- * If the guard hardens and it becomes difficult to shape it, soak it in hot water again, then shape it.

10. Make sure that the product is not loose or out of place. If you feel any discomfort, repeat the steps from the beginning to reattach the product.

The product correctly fastened (right hand)

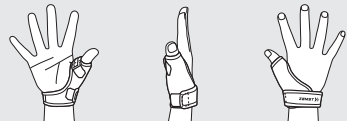<Palm of hand> <Side view> <Back of hand>

11. After the product has been molded, dry it in the shade.

Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Disclaimer:

- * The product specifications and appearance are subject to change without notice for the purpose of improvement.
- * When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.
- * We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmaz Co., Ltd.

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